


7

Illustrated Example of Taking a Picture and Adding Comments

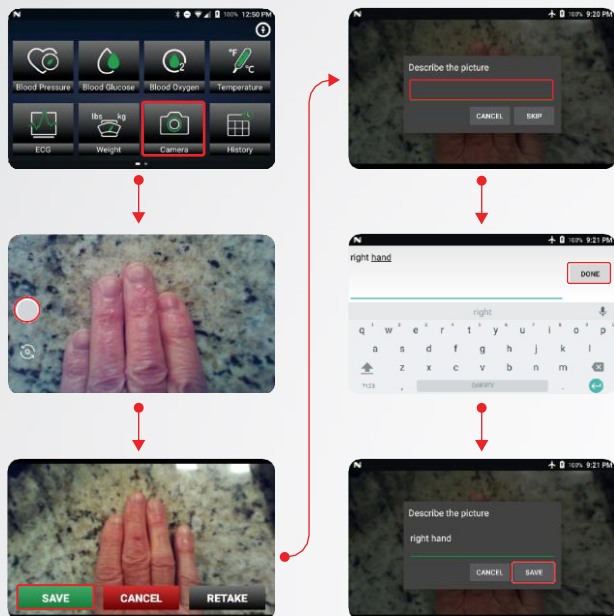

SAVE

Save the previous measurement in the device locally, then return to the home page

CANCEL

Cancel the previous measurement and **DO NOT** save any measuring result and return to the home page

RETEST

Cancel the previous measurement and restart the measuring process

vitals360⁺
Improving Efficiency

QUICK REFERENCE GUIDE

VoCare Vitals360[®]

Professional Grade Mobile Medical Device

Questions or Comments?
Call 1-855-756-6766
For More Information contact
sales@vocare.com



VoCare[™]
HEALTH MONITORING MADE SIMPLE

P: 317.973.1003
F: 317.873.2628

Corporate/International
49 Boone Village 222
Zionsville, IN 46077 USA

sales@vocare.com
vocare.com



This device was designed to capture and display vital sign results to healthcare providers and lay persons in a clinical or home environment.

The device is available only through prescription by a healthcare provider and is intended for spot-checking blood pressure, blood glucose (Not approved in USA), pulse, blood oxygen saturation, ECG, respiration rate, heart rate, infrared temperature, weight and height. This Quick Guide is a reference that does not substitute for training by a healthcare professional prior to use.

These vital sign measurements can be stored on the device for display or sent by way of a third-party Medical Device Data System (MDDS) platform.

The device natively does not perform any decision support functions or provide alerts to the user.

VoCare[™]
HEALTH MONITORING MADE SIMPLE

vitals360⁺
Improving Efficiency

1

Illustrated Example of Measuring Blood Pressure

3

Illustrated Example of Measuring Forehead Temperature

5

Illustrated Example of Manually Storing Height

2

Illustrated Example of Measuring Blood Oxygen

4

Illustrated Example of Measuring ECG

6

Illustrated Example of Manually Storing Weight